

# FFP2 NR RESPIRATOR

## Technical Specification Sheet Particle Filtering Half Mask (FFP2)

### DP-A-22-K & DP-A-22L-K

#### Key Features:

- CE Approval
- Reach Approval
- Flammability Rating Class 1
- Cup-shaped 3D curved face design
- Matches various face shapes
- Adjustable nose clip
- Available in 2 sizes for an optimal fit for smaller face
- Adjustable head bands / earloops

#### Material Composition:

- Straps- Adjusted braided elastic
- Nose clip – Aluminum
- Inner panel – PP non-woven
- Filter – Melt blown non-woven
- Shell – Polyester fibre cotton
- Front panel – Fluid resistant non-woven
- Approximate weight of product 16g

#### Time Use Limitation:

- One shift only

#### Approval & Standards:

- CE Approval
- EN 149: 2001 + A1:2009
- PFE (Sodium Chloride)  $\geq 94\%$
- PFE (Paraffin Oil)  $\geq 94\%$
- Reach Approval

#### Instructions for Use:

- Suitable for use against solid and liquid particulates including oil based mists.
- This product does not protect against gases or where oxygen levels are below 19.5%.
- Do not use when concentrations or contaminants are unknown or dangerous to life.
- Leave the work area if distress or dizziness occurs.
- Do not alter or modify the product in any way.
- This product is intended only for single use and must not be used for more than 8 hours. Replace mask if contaminated or if breathing becomes difficult.
- Facial hair may affect the efficiency of the product.
- Do not use in an explosive area.
- Ensure the product is suitable for the intended use.
- Place/protect the product when not in use within its packaging.

#### Limitations of use:

- Protection will only be offered if the mask is fitted correctly.
- The mask must be discarded and replaced if it is a). Removed whilst in the contaminated area, b). Breathing difficulties due to clogging are experienced or, c). The mask becomes damaged.
- The mask, when in contact with the skin may cause allergic reactions to susceptible individuals. If this occurs leave the hazard area, remove mask and seek medical advice.



#### Notes:

- Use in well ventilated areas only, which are not deficient in oxygen and do not contain explosive atmospheres.
- This mask is not designed to be worn by people with facial hair.

#### Cleaning & Maintenance:

- Not applicable – disposable product.
- Store the masks out of direct sunlight away from chemicals and in a dry atmosphere.

#### Fit Check:

- Check the fit of the mask by covering with both hands and exhale vigorously. Any leaks around the face seal must be eliminated before entering contaminated area.

#### Warning:

- Mask must be fitted in accordance with the instructions supplied.
- Care must be taken to ensure that the correct mask type is selected for the work to be carried out.
- If in doubt as to the type of mask required seek professional advice.